- EN** User instructions (*ENGLISH*)
- DE** Gebrauchsanleitung (*GERMAN / DEUTSCH*)
- ES** Instrucciones para el usuario (*SPANISH / Español*)
- PT** Instruções de utilização (*PORTUGUESE / Português*)
- IT** Istruzioni per l'utente (*ITALIAN / ITALIANO*)
- FR** Mode d'emploi (*FRENCH / FRANCAIS*)
- NL** Gebruiksaanwijzing (*DUTCH / NEDERLANDS*)
- DK** Brugsanvisning (*DANISH / DANSK*)
- SE** Bruksanvisning (*SWEDISH / SVENSKA*)
- NO** Brugsanvisninger (*NORWEGIAN / NORGE*)
- FI** Käyttöohjeet (*FINNISH / SUOMI*)
- PL** Instrukcja użycia (*POLISH/POLSKI*)

Chairside Technique Guide

Intra-oral Adjustment/Polishing for Zirconia and Lithium Disilicate


1 Start by gross adjusting and contouring using the blue coarse grit grinders. Use the flat end taper for occlusal/labial/lingual/buccal surfaces and the disc for interproximal surfaces.

2 Use the green medium grit polishers for smoothing and pre-polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.

3 Use the orange fine grit polishers for high gloss polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.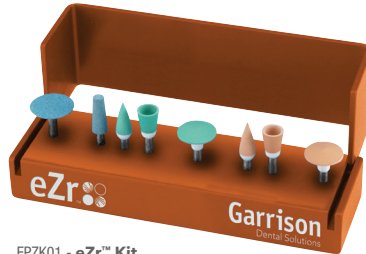

FPZK01 - eZr™ Kit

	Head Size (mm)	Grit	Usage	RPM
	11.5 x 3	Coarse	Step 1 - Gross removal interproximal surfaces	8,000 - 12,000 rpms
	4 x 10	Coarse	Step 1 - Gross removal occlusal/labial/lingual/buccal surfaces	8,000 - 102,000 rpms
	4 x 10	Medium	Step 2 - Smoothing and pre-polishing occlusal surfaces	7,000 - 10,000 rpms
	6 x 7.5	Medium	Step 2 - Smoothing and pre-polishing labial/lingual/buccal surfaces	7,000 - 10,000 rpms

	Head Size (mm)	Grit	Usage	RPM
	10 x 2.5	Medium	Step 2 - Smoothing interproximal surfaces	7,000 - 10,000 rpms
	4 x 10	Fine	Step 3 - High-gloss polishing occlusal surfaces	7,000 - 10,000 rpms
	6 x 7.5	Fine	Step 3 - High-gloss polishing labial/lingual/buccal surfaces	7,000 - 10,000 rpms
	10 x 2.5	Fine	Step 3 - High-gloss polishing interproximal surfaces	7,000 - 10,000 rpms

Company Headquarters

150 DeWitt Lane, Spring Lake, Michigan 49456 USA
 Toll-Free (US and Canada): 888.437.0032
 T 616.842.2244 F 616.842.2430
 gds@garrissidental.com garrissidental.com

Garrison

Instructions for Use

Intra-oral Adjustment/Polishing for Zirconia and Lithium Disilicate

1 Suitability for use

Visually check the polishers for cracks, chips and corrosion. Ensure that the polishing medium remains secure on the bur. Do NOT use if damaged or worn out.

2 Sterilization

Instruments are shipped non-sterile. Complete cleaning and sterilization instructions may be found at www.garrissidental.com/sterilization. Autoclave the instruments at 134°C with a holding time of five (5) minutes and a drying time of ten (10) minutes. It is the responsibility of the dentist or licensed practitioner to ensure effective sterilization.

3 Storage

Store the polishers in the sterilization container until required for use. Storage should be in dry, clean conditions at ambient temperature.

4 Precautions

- A.** The device is to be used by a dentist or licensed practitioner
- B.** Wear gloves when handling instruments
- C.** Handpieces should be maintained in good working order
- D.** Ensure the polisher is securely gripped in the handpiece collet
- E.** Wear eye protection to shield against particles
- F.** Wear a surgical mask to avoid inhalation of particles

5 Use

- A.** Maintain constant movement to avoid localized heating and potential damage
- B.** The speed range for use is 7000-10,000 RPM. Do NOT exceed 20,000 RPM
- C.** Avoid applying excessive pressure or leverage
- D.** Best results are achieved by using polishers in order from coarse to fine grit

6 Post Use

Used burs should be considered contaminated and require appropriate handling precautions for subsequent reprocessing.

For current instructions including cleaning and sterilization, please visit www.garrissidental.com

Intra-oral Adjustment/
 Polishing for Zirconia and
 Lithium Disilicate

Non-sterile, RX only, always use a rubber dam. Made Germany.

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© EVE Ernst Vetter GmbH, Neureutstr. 6, 75210 Kelttern, Alemania
EVE Ernst Vetter GmbH, Neureutstr. 6,