

PerForm Proximal Contact Instrument

Garrison

PerForm Proximal Contact Instrument

Garrison
Dental Solutions

Garrison

PerForm Proximal Contact Instrument

Tear open for instructions.

Tear open for instructions.

Garrison
Dental Solutions

888.437.0032

REORDER TODAY!
Toll-free 888.437.0032 • www.garrisdental.com

Non-sterile, RX only, always use a rubber dam.

For cleaning and sterilization instructions go to:
garrisdental.com/sterilization




USA Office
150 DeWitt Lane
Spring Lake, MI, USA 49456
P 616.842.2244 F 616.842.2430

Europe Office
Carlstrasse 50
52531 Uebach-Palenberg, Germany
P +49.2451.971.409



D. R.M. Green, Authorized Representative European
Compliance Services Ltd.
Oneplan Court, Pinecliff Square
Oxford, OX1 1BB, UK

10397-1113

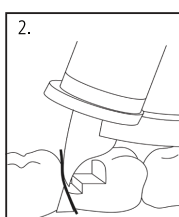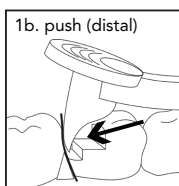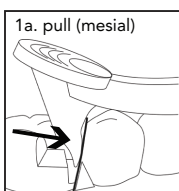

1. Place the first composite increment to the level of the pulpal floor.
2. Press the appropriate end of the PerForm instrument into the uncured composite increment and either push (distal) or pull (mesial) as indicated on the handle until desired contact is achieved. Do not pry! (figure 1a & 1b)
3. Place the curing light directly on the Fresnel lens and cure the composite while maintaining the mesial or distal pressure. (figure 2)
4. Remove the PerForm instrument and fill the small void left by the tip with flowable composite.