




Mini Polishers for Composite

These instructions are valid for all Rally Mini Polishers that are used for restorative and preventive work.

Single-use polishers are labeled with  and must not be used more than once, as multiple uses of these polishers can cause unnecessary risk to the patient and staff.

Composition: The polishers consist of synthetic/silicone rubber, pigments and abrasive. The shanks are made of medical plastic.

Indication: Rally Mini Polishers are used for polishing different materials which are specified composites & hybrid ceramics on the packaging label.

Contraindications: If a patient is known to be allergic to any of the above components, the polishers should not be used.

Adverse effect: None known to date.

General security advices:



Do not exceed
maximum speed



Wear breathing
protection



Wear eye pro-
tection



Use waterspray
minimum 50ml/min

Precautions:

- Products may only be used in the specified field of application.
- Check the safe connection of the polisher before using it and perform a test run.
- Use proper hand piece only and remove any damaged, bent, or no longer central running polishers immediately.
- Avoid excessive pressure which can cause tilting, bending or blocking of the polisher.
- Insufficient water cooling can result in irreversible damage to the tooth and the surrounding tissue.
- Always keep track of LOT numbers of polishers to ensure traceability.

Storage: A dry, dust-free storage at 2 - 28°C (36 - 82°F) is recommended. Improper storage conditions can shorten the shelf life and cause malfunction of the product.

Expiry date: See imprint on label.

REMOVE



Choose the blue polishers
for fast removal and
contouring

SMOOTH



Choose the red polishers
for smoothing and silky
shine

SHINE



Choose the gray polishers
for ultra high shine

USA Office

150 DeWitt Lane
Spring Lake, MI 49456
P 616.842.2244 F 616.842.2430

Europe Office

Carlstrasse 50
52531 Uebach-Palenberg, Germany
P +49.(0)2451.971.409 F +49.(0)2451.971.410

Keep out of reach of children. For use in dentistry only. Rx only. Caution: U.S. federal law restricts this device to be sold by or on the order of a licensed dentist.

Garrison

Dental Solutions

For complete cleaning and sterilization
instructions go to: [garrisondental.com/
sterilization](http://garrisondental.com/sterilization)

Made in Liechtenstein

1739r1116 rev B

DOC #: 1739r1116

REV #: B

DESCRIPTION: Rally IFUs

LANGUAGES: English

FILE TYPE: Adobe InDesign CS6

PREPARED BY: JLM

SCALE: 100%

INK: black ink 2 sides

PAPER STOCK: 50# White offset

NOTES: Flat size 3.5 x 4.5, Folds in half to 1.75 x 4.5. Then accordion folded to 1.75 x 1.5 with Rally logo facing outward