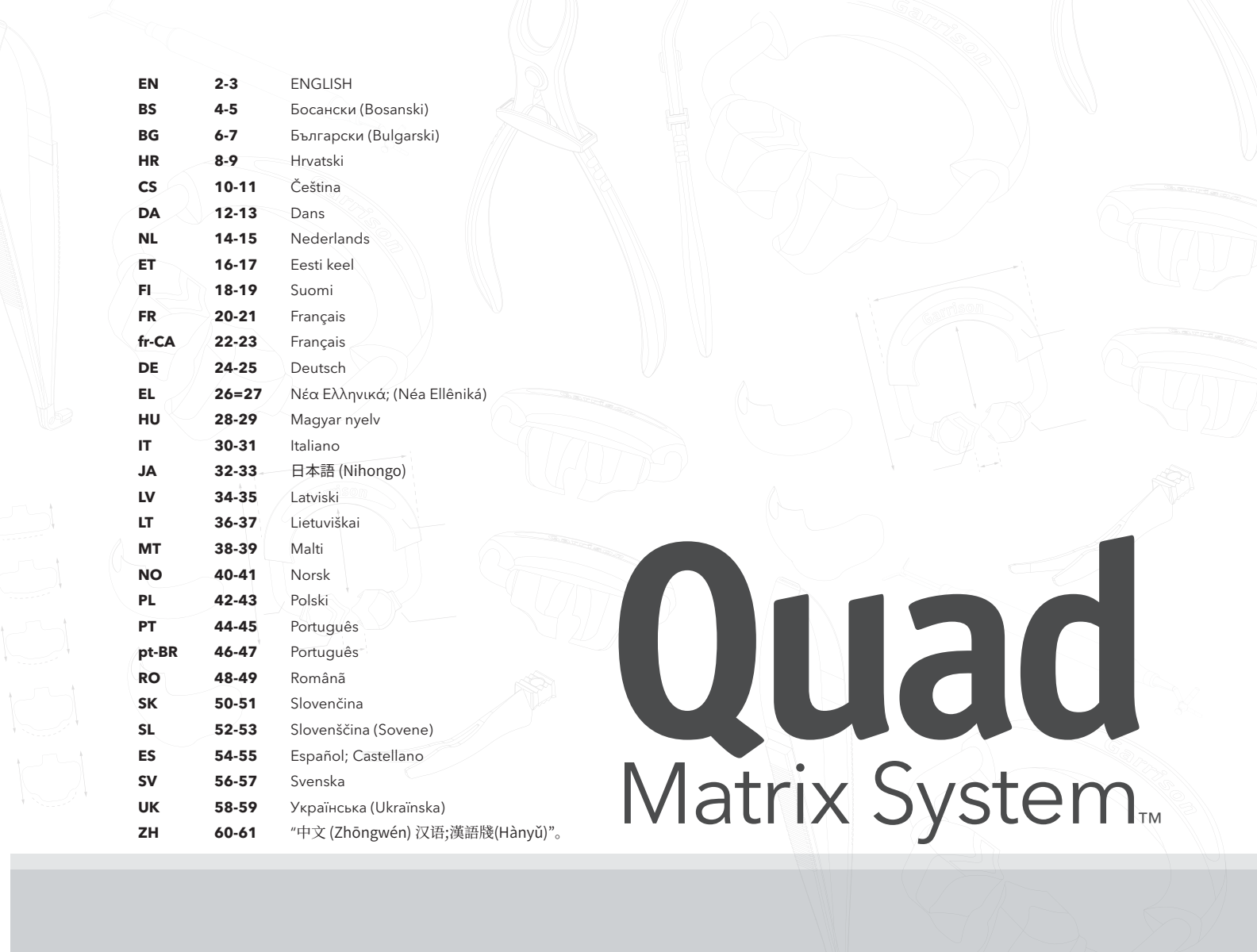
EN	2-3	ENGLISH
BS	4-5	Босански (Bosanski)
BG	6-7	Български (Bulgarski)
HR	8-9	Hrvatski
CS	10-11	Čeština
DA	12-13	Dans
NL	14-15	Nederlands
ET	16-17	Eesti keel
FI	18-19	Suomi
FR	20-21	Français
fr-CA	22-23	Français
DE	24-25	Deutsch
EL	26=27	Νέα Ελληνικά; (Néa Ellēniká)
HU	28-29	Magyar nyelv
IT	30-31	Italiano
JA	32-33	日本語 (Nihongo)
LV	34-35	Latviski
LT	36-37	Lietuviškai
MT	38-39	Malti
NO	40-41	Norsk
PL	42-43	Polski
PT	44-45	Português
pt-BR	46-47	Português
RO	48-49	Română
SK	50-51	Slovenčina
SL	52-53	Slovenščina (Sovene)
ES	54-55	Español; Castellano
SV	56-57	Svenska
UK	58-59	Українська (Ukrainska)
ZH	60-61	“中文 (Zhōngwén) 汉语;漢語 (Hànyǔ)”。

Quad Matrix System™

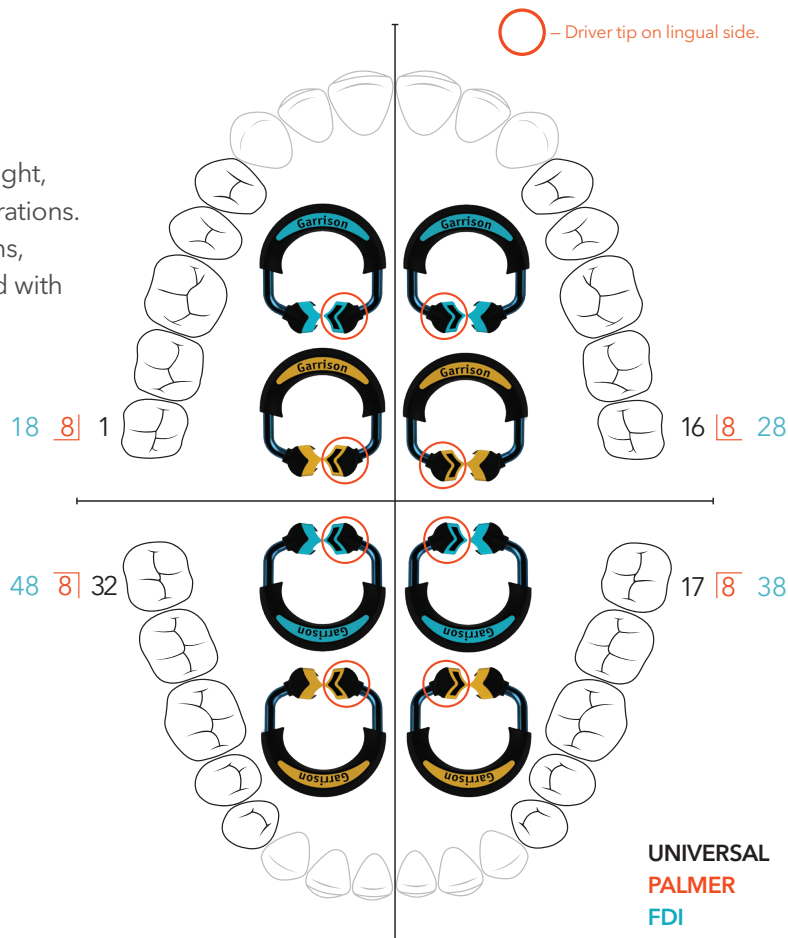
Quad Matrix System™

The Quad Matrix System is designed to create tight, anatomical contacts on Class II composite restorations. It is particularly effective for quadrant restorations, including back-to-back restorations when paired with the Firm Bands.

Ring Placement Guide:

Quad Matrix System rings feature an asymmetrical tip design. The Driver Tip is identified by an arrow printed on the top of the tip.

Recommended Driver Tip placement is indicated on this chart. The Driver Tip should be placed on the side with the deepest/widest embrasure, which is typically the lingual side.

For Patent information see www.garrisondental.com/patents

Non-sterile, RX only, always use a rubber dam.

For sterilization instructions, visit: garrisondental.com/sterilization.

Instructions for Use:

Single Restoration:

- 1. Place Rubber Dam:** Isolation of the area helps to prevent contamination from blood and saliva. Aspiration of small parts is a valid concern.
- 2. Prepare Tooth:** Prepare the tooth to remove all caries and unsupported tooth structure.
- 3. Place Sectional Matrix Band:** Select a matrix band that closely approximates the occlusogingival height of the tooth. The tab should be oriented toward the occlusal margin and may be bent over the adjacent tooth after placement (figure 1).
- 4. Insert Wedge:** Keep a finger on the matrix band to prevent the wedge from dislodging the band during insertion. Wedge from the buccal so that the split tip of the wedge is on the lingual (figure 2).
- 5. Apply Matrix Ring:** Select a retainer ring appropriate for the embrasure and tooth being restored. Spread it with the Garrison Premier Ring Placement forceps. Place the ring over the wedge with the asymmetrical driver tip between the ends of the split wedge (figure 3).
- 6. Ensure Proper Contact:** Push the matrix band firmly into contact with the adjacent tooth in the desired contact area.
- 7. Etching, Disinfecting, and Bonding:** Follow the manufacturer's instructions for your preferred etching, disinfecting, and bonding techniques.
- 8. Apply Composite Resin:** Apply your preferred composite resin following the manufacturer's instructions.
- 9. Remove Components:** Remove the ring, wedge, and matrix band.
- 10. Contour and Polish:** Contour and polish the restoration as needed.

figure 1

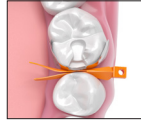
figure 2

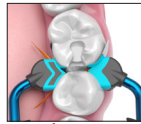
figure 3

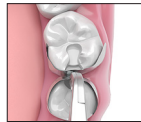
figure 4

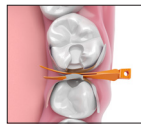
figure 5

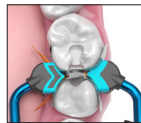
figure 6

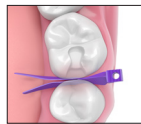
figure 7

Back-to-Back Simultaneous Restorations:

- 1. Place Rubber Dam:** Isolation of the area helps to prevent contamination from blood and saliva. Aspiration of small parts is a valid concern.
- 2. Prepare Teeth:** Prepare the teeth to remove all caries and unsupported tooth structure.
- 3. Place first Firm Band:** Select a Firm Band that most closely approximates the marginal ridge height of the tooth being restored. Place the band on the tooth with prep facing away from you. Carefully position the Firm Band in the interproximal space, ensuring that it extends past the gingival margin of the preparation.
- 4. Place the Quad wedge:** Place the wedge so the split ends are on the side of the widest embrasure, which typically is the lingual side. Use apical finger pressure on the top of the Firm Band to prevent it from shifting during wedge placement. Grip the Quad wedge placement block and have the longer of the two split ends go in first.
- 5. Place the second Firm Band:** When you place the second band, the wedge will prevent the band from seating all the way. To resolve this, put apical finger pressure on top of both bands and pull back the wedge slightly until the second firm matrix band sinks into place. Push the wedge back into position (Figures 4 and 5).
- 6. Apply Matrix Ring:** Select a retainer ring appropriate for the embrasure and tooth being restored. Spread it with the Garrison Premier Ring Placement forceps. Place the ring over the wedge with the asymmetrical driver tip between the ends of the split wedge (figure 6).
- 7. Ensure Proper Contact:** Push the matrix bands firmly into contact with each other in the desired contact area.
- 8. Etching, Disinfecting, and Bonding:** Follow the manufacturer's instructions for your preferred etching, disinfecting, and bonding techniques.
- 9. Apply Composite Resin:** Restore both teeth simultaneously, ensuring the composite resin is packed and cured properly for both restorations following the manufacturer's instructions.
- 10. Remove Components:** Remove the ring, wedge, and matrix bands.
- 11. Contour and Polish:** Contour and polish restorations as needed.

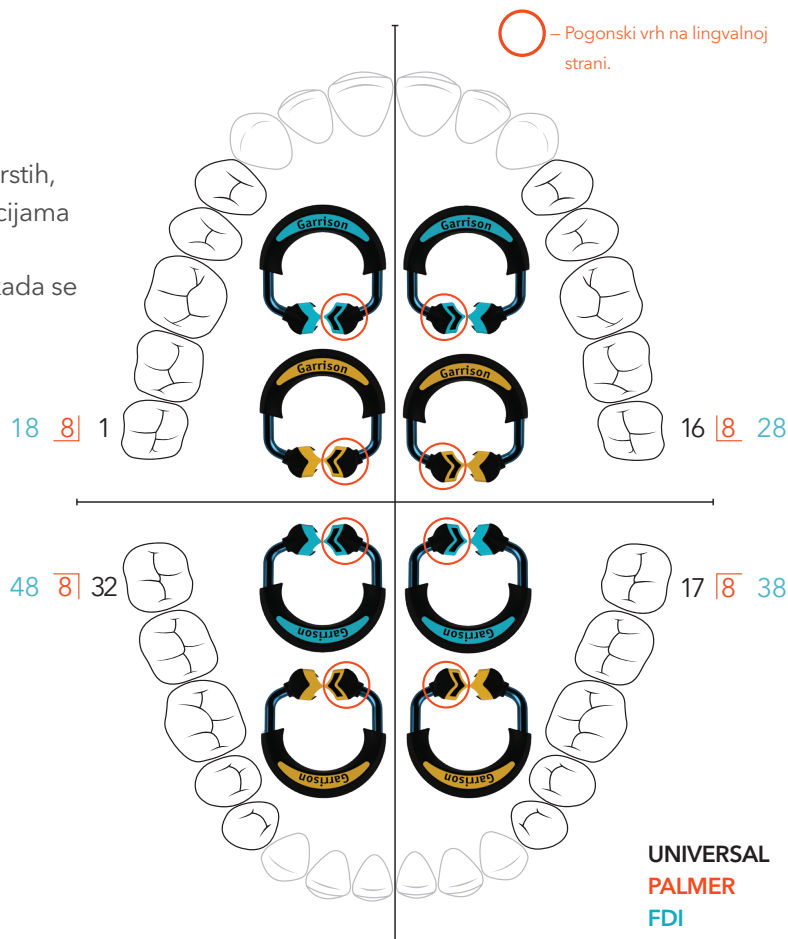
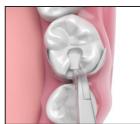
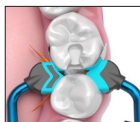
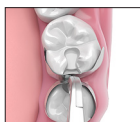
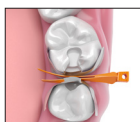
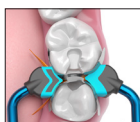
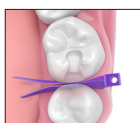
Special anatomical wedge: The purple anatomical wedge should be used when an interproximal concavity is present on the tooth surface. This will improve the seal of the matrix band against the tooth (figure 7).

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