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
User instructions

To Reorder:

(USA) Toll-free 844.200.0721, reinventdental.com
(Büro Deutschland) +49 (0) 2451 4095890, re-inventdental.de

- EN** For an online version of these instructions for use, scan the QR code on this page.
- DE** Für eine Gebrauchsanweisung in deutscher Sprache scannen Sie den QR-Code auf dieser Seite.
- ES** Para obtener instrucciones de uso en español, escanee el código QR en esta página.
- PT** Para obter instruções de uso em português, leia o código QR nesta página.
- IT** Per le istruzioni d'uso in Italiano, scannerizzare il codice QR di questa pagina.

Currently this IFU is NOT printed. It is for use on the website and for distribution as a pdf. If any changes are made to 1699r0715 they must also be made to this doc 1777r1118.

Languages translated: English, German, Portuguese, Spanish, and Italian

FILE NAME: 1777r1118 Nitin Instructions.indd

DOC #: 1777r1118

REV #: B

DESCRIPTION: Nitin Multi language Instructions IFUs

LANGUAGES: Multiple languages

FILE TYPE: Adobe Adobe InDesign 2023

PREPARED BY: JLM

SCALE: 100%

DIE LINE:

INK: CMYK

PAPER STOCK: 70# matte text, White, cutting, saddle bound

PRINTING: Head to head, bleed .25 on covers

NOTES: finished size 7.75 x 5.75

BLANK

User Instructions

The NiTin™ Sectional Matrix System is designed to create predictable contacts on Class II composite restorations. The matrix bands are pre-contoured for anatomically accurate restorations while premolar and molar separator rings ensure optimal tooth separation for tighter contacts.

Material Selection: Select a matrix band that most closely approximates the height of the tooth. Use a band with a sub-gingival extension for larger and/or deeper restorations.

The white ring is designed to provide the optimum separation force for premolars and small molars. The black ring will provide optimum separation for typical molars. The rings are designed so that the black ring can be stacked over the white ring for multi-tooth/surface restorations.

Typical Class II restorative technique:

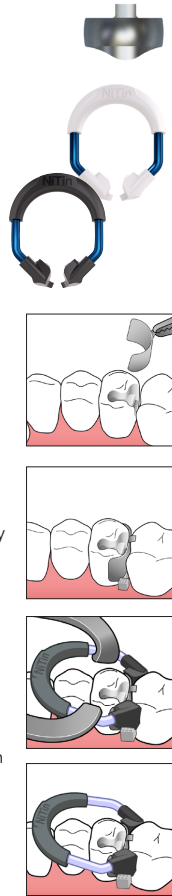
Place a rubber dam or similar isolation product. Isolation aids in the prevention of contamination from crevicular fluids as well as preventing aspiration of small parts.

Place matrix band. Grip the tab on the matrix band with the NiTin™ Band Forceps or cotton pliers and place interproximally with concave surface toward the tooth being restored. Once placed, bend the tab away from the tooth being restored.

Place wedge. Grip the block on the wedge with the NiTin™ Band Forceps, cotton pliers or hemostats. Place a finger on the matrix band to prevent it from becoming dislodged and insert the wedge firmly.

Place ring. Engage the ring wire into the grooves of the NiTin™ Ring Placement Forceps and open the ring just enough to allow placement in the interproximal space. Position the notched tips of the ring over the wedge seating them as far gingivally as possible.

Gently burnish matrix band. Ensure that the matrix band is


firmly in contact with the adjacent tooth.

Place restorative materials. Apply your preferred restorative materials following the manufacturer's instructions.

Remove the ring, wedge, and band. Removal of the matrix band may require the use of a hemostat due to the tight contacts achieved with the NiTin™ system.

Finish and polish. Use your preferred finishing and polishing technique to complete the restoration.

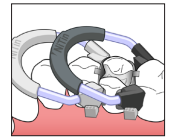MOD's and multiple tooth restorations. NiTin™ separator rings are designed so that the black ring can be stacked over the white ring to facilitate MOD or multiple tooth restorations. Place and wedge both matrix bands before placing rings. Place the mesial ring first then stack the distal ring over top.

Cleaning and Sterilization Instructions:

NiTin™ Matrix Rings maintain their strength and appearance by using the following cleaning and sterilization procedure: Remove debris with a soft bristle brush, using a mild detergent in warm water. **DO NOT** use solutions containing acetone or ketone. Steam autoclave sterilization only.

Sectional Matrix Bands are single use disposable items. Sterilization prior to use is recommended and can be performed using any common sterilization equipment or chemicals.

Instruments & Forceps may be dry heat sterilized, chemoclaved, steam autoclaved or cold sterilized following the sterilization unit manufacturer's instructions.

Wedges are single use, disposable items.

For complete sterilization instructions go to: reinventdental.com/sterilization or re-inventdental.de/sterilisation

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