ENGLISH

Adjustable FitStrip™ System

Garrison’s innovative FitStrips™ represent a breakthrough in restorative dentistry. Unlike other strips that rely on constant squeezing of the strip to maintain curvature, our FitStrips™ features an adjustable wheel to lock desired curvatures into place for maximum precision, maximum patient safety and minimal fatigue.

FitStrips™ is ideal for proximal contact adjustment, marginal crown seating, veneers, onlays, proximal contouring of composite fillings, and cement clean-up.

Loosening (8)

Tightening (9)

IPR – Interproximal Reduction

The Garrison FitStrip™ System makes interproximal reduction easy and fast due to the design of our patented removable handle. The handle allows you to work from outside the patients’ mouth creating more comfort for your patients. The twist of the wheel allows you to adjust the strip, tightening or loosening the strip as needed. This permits the doctor to accomplish their interproximal contouring and finishing requirements with minimal time, effort and the lack of fatigue.

When tightening the adjustment wheel, the strip will become inflexible for tooth contact areas. When it is loosened, the strip will curve allowing you to finish all of your contouring needs (10).

Single-Sided FitStrip™ System

Our versatile single-sided FitStrips™ is perfect for contouring and finishing composite restorations and for single tooth slenderization (11).

Double-Sided FitStrip™ system

Garrisons’ double-sided FitStrips™ excel at interproximal reductions, contouring and finishing composite restorations. For stripping and fast adjustments, it is recommended to tighten the strip and use with or without the handle. For contouring, loosen the strip (12).

Serrated FitStrip™ System

The Garrison sesrated strip is ideal for separating fused contacts of composites and cement (13).

“It is easier to reach tight posterior areas if you ave the handle attached.”

Contact Adjustment

The Super Fine Single Sided Strip is especially designed to adjust tight contact areas in Crown and Bridge restorations. By loosening the strip and creating a curve, the strip allows you to adjust the crown into perfect contact (14).

Technical Information

The FitStrip™ System cleaning and sterilization guide:

User information for the FitStrip™ System Diamond and Serrated Finishing Strips:

The adjustable FitStrip™ System is suitable for removing excess material and interproximal shaping and finishing.

Instructions for cleaning and sterilization:

1. Please use gloves for your safety.
2. The FitStrip™ should be brushed clean with water and soap, or a cleaning solution of your choice.
3. The FitStrip™ is suitable for ultrasonic baths, if appropriate clamping implements are used, which prevent instruments from touching.
4. Avoid excessive touching of the strip and ensure all debris is removed prior to use. Each strip must be washed and disinfected before every use.
5. Strips can be vapor-sterilized in an Autoclave with steam at 273°F (134°C) for 5 minutes.

Non-sterile

Tie one end of floss onto the Strip handle and tie the other end off on your finger, a dental dam clamp, or let it hang outside of the patient’s mouth.

For complete cleaning and sterilization instructions go to: garrisondental.com/sterilization

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ENGLISH
IPR Guide

Creating accurate interproximal spaces for orthodontic procedures, including the movement of minor teeth, has never been easier. Just use FitStrip™ diamond strips in different colors according to the recommended order.

Start the process of interproximal reduction (IPR) by beginning with the thinnest FitStrip and continue until the required space is achieved. The gauge should be used to check the interproximal space before proceeding to the next step of progression.

To achieve the best clinical outcome when aiming for an ideal anatomical contour in the posterior area, use the FitStrip in its curved form in the final step. It is best to adjust the strip (straight or curved) to the anatomical shape of the tooth being worked on.

The system can be used with almost any tooth anatomy and in combination with any system where precise interproximal spaces may be needed (e.g., Six Month Smiles™, MTM™, Invisalign™, Clear Correct™, etc.).

Step 1
Double-sided coated
0.11 mm

Step 2
Double-sided coated
0.21 mm

Step 3
Double-sided coated
0.30 mm

Step 4
Double-sided coated
0.40 mm

Step 5
Double-sided coated
0.50 mm

This guide serves as a starting point for approximate enamel reduction. Refer to the instructions for your individual system for technical questions.

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