



# Instructions for Use

Intra-oral Adjustment/Polishing for Zirconia and Lithium Disilicate

## 1 Suitability for use


Visually check the polishers for cracks, chips and corrosion. Ensure that the polishing medium remains secure on the bur. Do NOT use if damaged or worn out.

## 2 Sterilization

Instruments are shipped non-sterile. Complete cleaning and sterilization instructions may be found at [www.garrisondental.com/sterilization](http://www.garrisondental.com/sterilization). Autoclave the instruments at 134°C with a holding time of five (5) minutes and a drying time of ten (10) minutes. It is the responsibility of the dentist or licensed practitioner to ensure effective sterilization.

## 3 Storage

Store the polishers in the sterilization container until required for use. Storage should be in dry, clean conditions at ambient temperature.

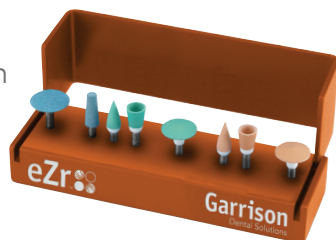

## 4 Precautions

- A. The device is to be used by a dentist or licensed practitioner
- B. Wear gloves when handling instruments
- C. Handpieces should be maintained in good working order
- D. Ensure the polisher is securely gripped in the handpiece collet
- E. Wear eye protection to shield against particles
- F. Wear a surgical mask to avoid inhalation of particles

## 5 Use

- A. Maintain constant movement to avoid localized heating and potential damage
- B. The speed range for use is 7000-10,000 RPM  
Do NOT exceed 20,000 RPM
- C. Avoid applying excessive pressure or leverage
- D. Best results are achieved by using polishers in order from coarse to fine grit

## 6 Post Use

Used burs should be considered contaminated and require appropriate handling precautions for subsequent reprocessing.

For current instructions including cleaning and sterilization, please visit **[www.garrisondental.com/sterilization](http://www.garrisondental.com/sterilization)**