- EN** Chairside Technique Guide *(ENGLISH)*
- DE** Für die Behandlung am Patientenstuhl *(GERMAN / DEUTSCH)*
- ES** Guía técnica para la consulta *(SPANISH / Español)*
- PT** Guia de Técnica de Ajuste/Polimento *(PORTUGUESE / Português)*
- IT** Guida tecnica alla poltrona *(ITALIAN / ITALIANO)*
- FR** Guide Technique au fauteuil *(FRENCH / FRANCAIS)*
- NL** Technische handleiding Chairside *(DUTCH / NEDERLANDS)*
- DK** Teknikguide til brug ved stolen *(DANISH / DANSK)*
- SE** Klinisk bruksanvisning *(SWEDISH / SVENSKA)*
- NO** Praktisk brukerveiledning *(NORWEGIAN / NORGE)*
- FI** Kliinisen työn opas *(FINNISH / SUOMI)*
- PL** Przewodnik techniczny dla użytkownika *(POLISH/POLSKI)*

Chairside Technique Guide

Intra-oral Adjustment/Polishing of Zirconia and Lithium Disilicate


- 1** Start by gross adjusting and contouring using the blue coarse grit grinders. Use the flat end taper for occlusal/labial/lingual/buccal surfaces and the disc for interproximal surfaces.

- 2** Use the green medium grit polishers for smoothing and pre-polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.

- 3** Use the orange fine grit polishers for high gloss polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.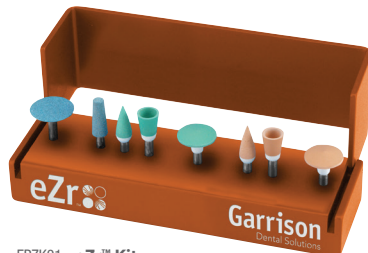

FPZK01 - eZr™ Kit

	Head Size (mm)	Grit	Usage	RPM
FPZC030	11.5 x 3	Coarse	Step 1 - Gross removal interproximal surfaces	8,000 - 12,000 rpms
FPZC010	4 x 10	Coarse	Step 1 - Gross removal occlusal/labial/lingual/buccal surfaces	8,000 - 102,000 rpms
FPZM020	4 x 10	Medium	Step 2 - Smoothing and pre-polishing occlusal surfaces	7,000 - 10,000 rpms
FPZM040	6 x 7.5	Medium	Step 2 - Smoothing and pre-polishing labial/lingual/buccal surfaces	7,000 - 10,000 rpms

	Head Size (mm)	Grit	Usage	RPM
FPZM030	10 x 2.5	Medium	Step 2 - Smoothing interproximal surfaces	7,000 - 10,000 rpms
FPZF020	4 x 10	Fine	Step 3 - High-gloss polishing occlusal surfaces	7,000 - 10,000 rpms
FPZF040	6 x 7.5	Fine	Step 3 - High-gloss polishing labial/lingual/buccal surfaces	7,000 - 10,000 rpms
FPZF030	10 x 2.5	Fine	Step 3 - High-gloss polishing interproximal surfaces	7,000 - 10,000 rpms

1 Suitability for use

Visually check the polishers for cracks, chips and corrosion. Ensure that the polishing medium remains secure on the bur. Do NOT use if damaged or worn out.

2 Sterilization

Instruments are shipped non-sterile. Complete cleaning and sterilization instructions may be found at www.garrisdental.com/sterilization. Autoclave the instruments at 134°C with a holding time of five (5) minutes and a drying time of ten (10) minutes. It is the responsibility of the dentist or licensed practitioner to ensure effective sterilization.

3 Storage

Store the polishers in the sterilization container until required for use. Storage should be in dry, clean conditions at ambient temperature.

4 Precautions

- A.** The device is to be used by a dentist or licensed practitioner
- B.** Wear gloves when handling instruments
- C.** Handpieces should be maintained in good working order
- D.** Ensure the polisher is securely gripped in the handpiece collet
- E.** Wear eye protection to shield against particles
- F.** Wear a surgical mask to avoid inhalation of particles

5 Use

- A.** Maintain constant movement to avoid localized heating and potential damage
- B.** The speed range for use is 7000-10,000 RPM. Do NOT exceed 20,000 RPM
- C.** Avoid applying excessive pressure or leverage
- D.** Best results are achieved by using polishers in order from coarse to fine grit

6 Post Use

Used burs should be considered contaminated and require appropriate handling precautions for subsequent reprocessing.

For current instructions including cleaning and sterilization, please visit www.garrisdental.com

Intra-oral Adjustment/
Polishing for Zirconia and
Lithium Disilicate

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