Daring WHITE™

Professional Teeth Whitening

Patent Pending

Caution: Regulations restrict dispensing of this product. To be sold only to a licensed dental professional.

DRM Green
European Compliance Services Ltd
Beaver House, 23-38 Hythe Bridge St, Oxford OX1 2EP, UK

European Office
Garrison Dental Solutions
Carlstrasse 50
D-52531 Uebach-Palenberg
Germany
+49.2451.971.409

www.daringwhite.com

Headquarters
Garrison Dental Solutions
150 DeWitt Lane
Spring Lake, MI 49456
USA

Garrison
Dental Solutions
www.daringwhite.com

Manufactured for Garrison Dental Solutions

1757/0118rev A Made in China

Daring[™]WHITE

Professional Teeth Whitening

Complete Application Instructions
Instrucciones completas para su aplicación
Instructions complètes d'application
Vollständige Anwendungsanleitung
Istruzioni Complete dell'applicazione
Kompletní návod k použití
Komplet Påføringsvejledning
Täydelliset käyttöohjeet
Teljes Alkalmazási Utasítás
Complete applicatie-instructies
Fullstendige instruksjer
Instruções de Utilização Completa
Instruktioner

15% Carbamide Peroxide Whitening Film
(5.5% Released Hydrogen Peroxide)

Garrison
Dental Solutions
www.daringwhite.com

Description

Daring White™ whitening films are the most comfortable, patient friendly whitening system available today. These films are designed to adhere to teeth for 1 to 2 hours to take advantage of the proprietary carbamide peroxide composition.

PLEASE NOTE: removal before 30 minutes can be done, but is more difficult, may tear and come off in pieces.

Counter Indications

- Do not smoke during whitening treatment.
- Avoid contact with eyes. If contact occurs flush with copious amounts of water.
- Avoid direct contact of the active surface of the film with the gums and/or salivary flow.
- Not for use by women who are pregnant or nursing.
- Not for use by children under 18 years of age.
- Avoid contact with clothing.
- Do not swallow the film.
- If irritation (such as redness, swelling, soreness) of the gum or mouth occurs, discontinue use and consult a dentist.
- If you brux or grind your teeth, do not wear overnight.
- Do not use if allergic to peroxides like hydrogen peroxide.
- Whitening will not affect porcelain crowns and veneers tooth colored acrylic restorations or amalgam.

Storage & Handling

- Keep the product refrigerated to maintain maximum shelf life.
- Do not store above 40°F (5°C). Do not freeze. Best use before date on label.
- Do not use if the packaging has been damaged, or if the seals are found to be broken or open.
- 15% initial concentration of carbamide peroxide. The concentration may change due to the nature of peroxide and this unique formulation.

Shelf Life: 36 months

Attention - Refer to the instructions for use and complete information.

Oxidizer - Avoid contact with reactive substances.


Storage Temperature - Store product at the indicated temperature.

R_p

To be sold only to dental practitioners.


For one-time use only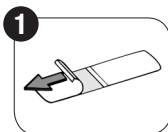

Remove paper exposing the whitening side of the film as shown.

Center film and place straight edge against teeth at gum line.

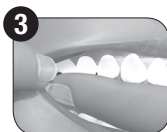Pull cheek away and push film onto back teeth.

Fold notched edge of film to back side of teeth.

To remove, grasp one corner and pull down away from front of teeth, then push film away from back of teeth.

Brush or rinse away any remaining gel. Rinse to sink.

Disclaimer: Garrison Dental Solutions believes this information to be accurate and is offered only for the benefit of its customers for use of the product under prescribed conditions. This document is not to be considered a warranty or guarantee of product performance, and Garrison Dental Solutions is not legally bound to such claims based on this document.

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