



# SIMPLE TECHNIQUE:

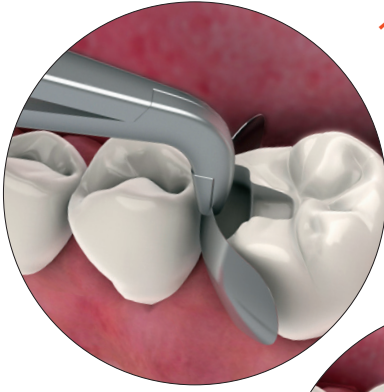


## 1. PLACE MATRIX BAND.

Select a band that matches the height of the tooth, give it a little extra curl with your fingers and place it with the concave edge towards the occlusal margin.

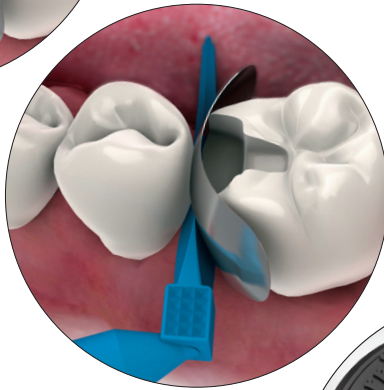

## 2. PLACE THE WEDGE.

Place a finger on the band to prevent dislodgment and firmly wedge to seal the gingival margin.

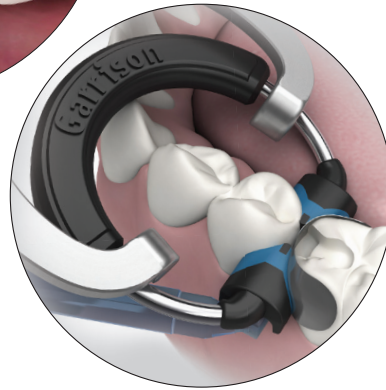

## 3. PLACE THE RING.

Open the ring with Garrison Ring Placement Forceps and place it directly over the wedge, seating it as far gingivally as possible.

## 4. PREPARE YOURSELF FOR THE BEST CLASS II'S EVER!