

## CASE STUDY

---

Class IIs next to the canine  
are easier with Strata-G™



Dr. Eduard  
**Epure**

Dr. Eduard Epure graduated from the University of Montreal in 2013, and went on to obtain a Master's degree in Operative Dentistry at the University of North Carolina. His mentors at this institution literally wrote the book of operative dentistry. Back in Quebec, he taught operative dentistry as a professor at the Faculty of Dentistry of the University of Montreal for six years. He currently acts as an articling supervisor for the Ordre des Dentistes, an expert for the SAAQ, and an examiner for the NDEB while maintaining a full-time private practice in the Montreal area.

---

**Garrison.**

150 DeWitt Lane | Spring Lake, MI 49456 USA  
toll-free 888.437.0032 | [www.garrisondental.com](http://www.garrisondental.com)



MO prep on tooth #5 (1.4) with selective removal of the previous defective composite restoration.



Matrix placement : a gap is visible until the wedge is placed.



Wedge placement and good seal obtained.



Short ring placement and visual confirmation that the matrix is well adapted to the adjacent proximal surface.



The ring shape enables stable placement on the mesial of the first premolar, which is usually a challenging situation for sectional matrix systems.



Composite restoration immediately after insertion and ring removal.



Buccal and lingual embrasures are anatomically shaped and require almost no finishing.



Minor finishing is required on the mesio-buccal embrasure to create a round transition between the composite and emergence profile.



After finishing and polishing.

# Garrison.

150 DeWitt Lane | Spring Lake, MI 49456 USA  
toll-free 888.437.0032 | [www.garrisondental.com](http://www.garrisondental.com)