



IPR Guide

Creating accurate interproximal spaces for orthodontic procedures, including minor tooth movement, has never been easier. Simply use the FitStrip™ color-coded diamond-impregnated strips according to the order provided.

Start the interproximal reduction (IPR) process with the thinnest FitStrip and continue in progression until the necessary space has been attained. **The measurement gauge should be used to check the interproximal space before moving to the next step in the progression.**

For the best clinical outcome when an ideal posterior anatomical contour is desired, finish your procedure using the FitStrip locked in the curved position. Preferably, you should base your choice of straight or curved position on the anatomical shape of the tooth you intend to reduce.

The system may be used on just about any dentition and in concert with any system where exact interproximal spaces may be required (Six Month Smiles™, MTM™, Invisalign™, Clear Correct™, among others).

IPR Guide

This guide is provided as a basic starting point for interproximal reduction. Refer to your individual system's Instructions For Use for technical guidance.



Step 1
Double-Sided
.11 mm



Step 2
Double-Sided
.21 mm



Step 3
Double-Sided
.30 mm



Step 4
Double-Sided
.40 mm



Step 5
Double-Sided
.50 mm



IPR Gauge
6 gauges
.1 - .5 mm Thick

Garrison
Dental Solutions

Company Headquarters
Toll-Free (US and Canada): 888.437.0032
gds@garrissidental.com
garrissidental.com

European Office
info@garrissidental.net
garrissidental.com
Made in Israel.

© 2018 Garrison Dental Solutions, LLC

1772r0818 rev A