



Flexible
Interproximal finishers with a
Twist

New Subgingival Trimmers and Finishers!

Subgingival FitStrips are perfectly designed to remove composite overhangs and excess cement quickly and safely from hard-to-reach subgingival areas. These strips are 33% more narrow at only 2.5 mm in height versus standard FitStrips, and with their center, non-abrasive Safety Zone, slip easily between teeth and below the contact area. The medium, fine and super fine durable diamond coating makes quick work of the cleanup and provides a silky-smooth finish.



One handy handle!

FitStrip's simple, attachable handle makes a world of difference for both clinician ease-of-use and patient comfort.

Ready for a marathon!

FitStrips are very durable and long lasting. They stay sharp and ready to go even after multiple autoclave cycles.

Starts bold — finishes clean

1. Medium grit for fast removal
2. Fine grit smooths things out
3. Super fine strips put the perfect finishing touch on things

Narrow focus

- 33% more narrow than standard FitStrips
- Only 2.5 mm in height

Stay in the zone

- Non-abrasive Safety Zone protects contact area
- Super-thin – only 0.05 mm!
- Slips easily and safely between teeth

See our
video online!

Garrison
Dental Solutions

FitStrip™

Flexible
Interproximal finishers with a
Twist



NEW!

FitStrip™ Subgingival Finishing and Single Sided Kit

FPSK08 contains:

2 handles

10 Strips: 2 subgingival super fine, 2 subgingival fine,
1 subgingival medium, 1 serrated saw,
1 each single sided .08 mm, .10 mm, .13 mm, .18 mm

**10% off
your first
FitStrip™
order!**

Kits All kits come in a package of 10 FitStrips with 2 handles.

NEW Subgingival Finishing Kit

3 subgingival super fine, 3 subgingival fine, 3 subgingival medium,
1 serrated saw

FPSK07 \$179

NEW Subgingival Finishing and Single Sided Kit

2 subgingival super fine, 2 subgingival fine, 1 subgingival medium,
1 serrated saw, 4 single sided (1 of each .08 mm, .10 mm, .13 mm, .18 mm)

FPSK08 \$179

Simple Subgingival Technique:

1. Slide between teeth. Starting with the medium grit (blue), twist the colored barrel until the strip is locked straight. Position the Safety Zone in the affected interproximal space. With gentle apical pressure and a slight back-and-forth motion, slide the strip between the teeth*. The barrel may be twisted into a curved position at this time to allow the strip to follow the natural tooth contours.
2. Remove the overhang/excess cement. Move Fitstrip™ in a gentle buccal-lingual back-and-forth motion. Remove FitStrip™ and test the area with floss to ensure the desired result. Repeat as necessary.
3. Finish and polish. Place the fine grit FitStrip™ (red) into the interproximal space using the method in step one. Move it in a gentle buccal-lingual back-and-forth motion three or four times. Remove the fine strip and repeat the process with the super fine (yellow) for an ideal polish.

*If the teeth are bonded together or the contact is too tight to get the 0.05 mm Safety Zone between the teeth, use the FitStrip™ serrated saw (gray barrel) first to gain access.

Refills		Single Sided	Double Sided	10 pk w/2 handles	4 ct packs (add - 4 to model #)
Super fine 15 Micron		FPSXFSS 0.08 mm thick	FPSXFDS 0.11 mm thick	Either Refill \$179	\$85
Fine 30 Micron		FPSFSS 0.10 mm thick	FPSFDS 0.15 mm thick	Either Refill \$179	\$85
Medium 46 Micron		FPSMSS 0.13 mm thick	FPSMDS 0.21 mm thick	Either Refill \$179	\$85
Coarse 90 Micron		FPSCSS 0.18 mm thick	FPSCDS 0.30 mm thick	Either Refill \$179	\$85
Serrated Saw		FPSSAW 0.05 mm thick		Refill \$179	\$85
Super fine 15 Micron		FPSGXFSS 0.08 mm thick	N/A	Refill \$179	\$85
Fine 30 Micron		FPSGFSS 0.10 mm thick	N/A	Refill \$179	\$85
Medium 46 Micron		FPSGMSS 0.13 mm thick	N/A	Refill \$179	\$85
FitStrip replacement handles (2-pack) FPSHANDLE-2				\$30	

Be sure to check out the website at www.garrisondental.com for info on other innovative Garrison products, demo videos and our 6-month money-back guarantee.

Garrison
Dental Solutions

150 DeWitt Lane
Spring Lake, MI USA
49456

Offer expires 12/31/2021
To order: Call toll free **888.437.0032**
or online at www.garrisondental.com



Mention Code
IN121FSsubG





Flexible
Interproximal finishers with a
Twist

New Subgingival Trimmers and Finishers!

Subgingival FitStrips are perfectly designed to remove composite overhangs and excess cement quickly and safely from hard-to-reach subgingival areas. These strips are 33% more narrow at only 2.5 mm in height versus standard FitStrips, and with their center, non-abrasive Safety Zone, slip easily between teeth and below the contact area. The medium, fine and super fine durable diamond coating makes quick work of the cleanup and provides a silky-smooth finish.



One handy handle!

FitStrip's simple, attachable handle makes a world of difference for both clinician ease-of-use and patient comfort.

Do the Twist! Twist the color-coded barrel to adjust FitStrips' curvature and it's automatically locked in place.

Ready for a marathon!

FitStrips are very durable and long lasting. They stay sharp and ready to go even after multiple autoclave cycles.

Starts bold — finishes clean

1. Medium grit for fast removal
2. Fine grit smooths things out
3. Super fine strips put the perfect finishing touch on things

Narrow focus

- 33% more narrow than standard FitStrips
- Only 2.5 mm in height

Stay in the zone

- Non-abrasive Safety Zone protects contact area
- Super-thin – only 0.05 mm!
- Slips easily and safely between teeth

See our
video online!

Garrison
Dental Solutions

FitStrip™

Flexible
Interproximal finishers with a
Twist



NEW!

FitStrip™ Subgingival Finishing and Single Sided Kit

FPSK08 contains:

2 handles

10 Strips: 2 subgingival super fine, 2 subgingival fine,
1 subgingival medium, 1 serrated saw,
1 each single sided .08 mm, .10 mm, .13 mm, .18 mm

Kits All kits come in a package of 10 FitStrips with 2 handles.

NEW Subgingival Finishing Kit

3 subgingival super fine, 3 subgingival fine, 3 subgingival medium,
1 serrated saw

FPSK07 \$237

NEW Subgingival Finishing and Single Sided Kit

2 subgingival super fine, 2 subgingival fine, 1 subgingival medium,
1 serrated saw, 4 single sided (1 of each .08 mm, .10 mm, .13 mm, .18 mm)

FPSK08 \$237

Simple Subgingival Technique:

1. Slide between teeth. Starting with the medium grit (blue), twist the colored barrel until the strip is locked straight. Position the Safety Zone in the affected interproximal space. With gentle apical pressure and a slight back-and-forth motion, slide the strip between the teeth*. The barrel may be twisted into a curved position at this time to allow the strip to follow the natural tooth contours.
2. Remove the overhang/excess cement. Move Fitstrip™ in a gentle buccal-lingual back-and-forth motion. Remove FitStrip™ and test the area with floss to ensure the desired result. Repeat as necessary.
3. Finish and polish. Place the fine grit FitStrip™ (red) into the interproximal space using the method in step one. Move it in a gentle buccal-lingual back-and-forth motion three or four times. Remove the fine strip and repeat the process with the super fine (yellow) for an ideal polish.

*If the teeth are bonded together or the contact is too tight to get the 0.05 mm Safety Zone between the teeth, use the FitStrip™ serrated saw (gray barrel) first to gain access.

Refills		Single Sided	Double Sided	10 pk w/2 handles	4 ct packs (add - 4 to model #)
Super fine 15 Micron		FPSXFSS 0.08 mm thick	FPSXFDS 0.11 mm thick	Either Refill \$237	\$112
Fine 30 Micron		FPSFSS 0.10 mm thick	FPSFDS 0.15 mm thick	Either Refill \$237	\$112
Medium 46 Micron		FPSMSS 0.13 mm thick	FPSMDS 0.21 mm thick	Either Refill \$237	\$112
Coarse 90 Micron		FPSCSS 0.18 mm thick	FPSCDS 0.30 mm thick	Either Refill \$237	\$112
Serrated Saw		FPSSAW 0.05 mm thick		Refill \$237	\$112
Super fine 15 Micron		FPSGXFSS 0.08 mm thick	N/A	Refill \$237	\$112
Fine 30 Micron		FPSGFSS 0.10 mm thick	N/A	Refill \$237	\$112
Medium 46 Micron		FPSGMSS 0.13 mm thick	N/A	Refill \$237	\$112
FitStrip replacement handles (2-pack) FPSHANDLE-2				\$41	

*Canadian MSRP

Be sure to check out the website at www.garrisondental.com for info on other innovative Garrison products, demo videos and our 6-month money-back guarantee.

Garrison
Dental Solutions

150 DeWitt Lane
Spring Lake, MI USA
49456

Offer expires 12/31/2021
To order: Call toll free **888.437.0032**
or online at www.garrisondental.com



Mention Code
INCA121FSsubg

